

Alveolar Ridge Preservation after Tooth Extraction + Transcrestal Sinus Augmentation

Dr. Homa Zadeh
Woodland Hills, CA
United States



Process & Conclusion

Extraction and alveolar ridge preservation (ARP) of tooth #3 using Inter-Oss collagen and protecting the socket opening with InterCollage® Guide plus Platelet Rich Fibrin. This allowed for open healing in order to preserve mucosal tissues. After 5 months of healing, the ridge contour was well preserved.



Pre-operative view.




Pre-operative view.



Pre-operative X-ray.



Placement of InterOss® Collagen.

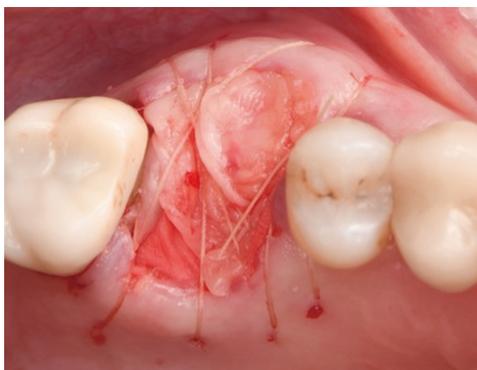

Placement of InterCollage® Guide and fixation.




Follow-up at 5 months.



Pre-implant X-ray.




Placement of implant.

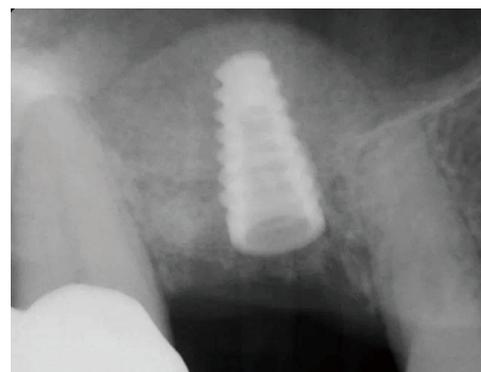


Post-implant X-ray.



Implant crown restoration.

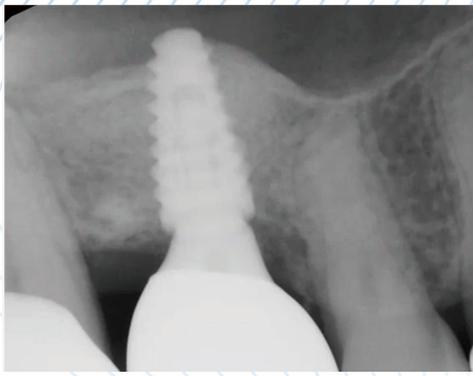

Post-restoration X-ray.

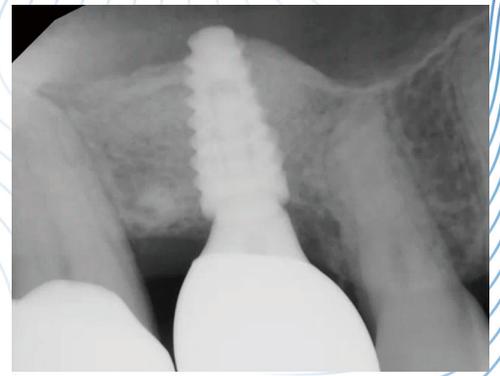

Follow-up at 1 year.